

NOUVA *Derm*

Advanced Skin Resurfacing
for Scalp, Face, and Body



AESTHETIC
MANAGEMENT PARTNERS

NOUVADerm

Unveil, Reclaim, and Restore Brighter, More Beautiful Skin

Enhance your practice with the NOUVADerm 1927nm Thulium Laser, a transformational skin resurfacing platform crafted to revitalize and renew the scalp, face and body. What distinguishes NOUVADerm is its ability to deliver non-ablative, sub-ablative and ablative pulses, offering a comprehensive and, versatile solution for today's most sought-after aesthetic treatments from unwanted pigment to fine lines and wrinkles, acne scars, stretch marks, skin tone and texture. NOUVADerm's short pulses can also painlessly prepare and support the hair & scalp.

Powerful, fast, versatile. NOUVADerm sets a new standard in ensuring optimal results and patient satisfaction.



Mechanism of Action




The NOUVADerm 1927nm Thulium laser harnesses the body's natural hydration by specifically targeting water within the epidermal and superficial dermal

layers. It delivers precise energy in a grid-like pattern, creating fractional microthermal zones (MTZ) that address specific skin concerns while preserving surrounding tissue to promote faster healing. The controlled micro damage stimulates the production of new collagen and elastin, leading to improved skin tone and texture.

The micro-injuries created by NOUVADerm typically penetrate the skin to a depth of >300 micrometers (μm), allowing for effective skin revitalization without excessive downtime.

NOUVADerm also provides practitioners the ability to tailor the procedures from non-ablative to ablative modes through customizable parameter settings based on patient needs and preferred downtime.

Controlling the Degree of Ablation

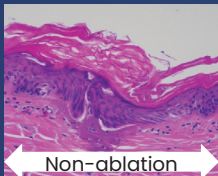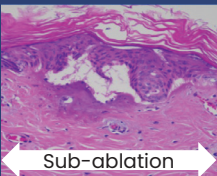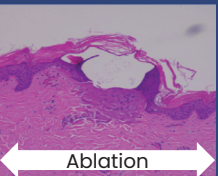20mJ 2000ms / 10W / 1 pulse	20mJ 4000ms / 5W / 1 pulse	40mJ 2000ms / 10W / 2 pulses
		
Non-ablation	Sub-ablation	Ablation
The histology shows mild damage to the epidermal layer with tissue coagulation in the upper dermis	This histology shows that the epidermal cells remain intact while separation of tissue occurred at the dermal epidermal junction with thermal effects on the dermal tissue.	This histology shows full ablation with the destruction of the epithelial cells and greater thermal effect in the dermis.

A Brighter Look for Your Patients and Your Practice

The NOUVADerm Laser is designed for flexibility, allowing treatments to be customized to each patient's unique skin type, condition, and goals. Practitioners can effortlessly switch between non-ablative, sub-ablative and ablative modes, providing precise control over treatment intensity and depth. The 7 preset modes that include the signature Hair Support mode for scalp treatment & hair support, and the 3 unique NOUVAGlo Modes for skin revitalization, offer unique procedures for your patients and your practice. NOUVADerm's tailored, more targeted approach from head to toe delivers unparalleled results and long-term patient satisfaction.

The NOUVADerm Advantage



Power and Speed

- The most powerful 1927nm laser on the market with 120mJ of power.
- Treats a face in 6 minutes with the largest 30mm spot size.
- Offers continuous lasing for faster, more even coverage for improved outcomes.




Advanced Aesthetic Solution

- Fractional delivery for faster recovery.
- Ablative, Sub-Ablative and Non-Ablative modes for customized treatments.
- Signature modes: NouvaGlo and Hair Support for customized treatments.




Elevated Experience

- Optimized skin revitalization for face and body.
- Unparalleled safety and efficacy.
- Increased patient throughput.

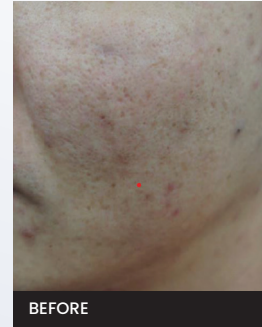


BEFORE

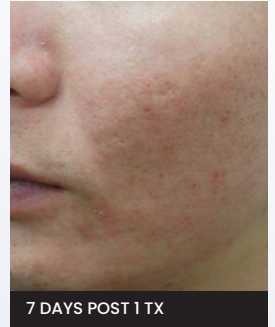


7 DAYS POST 1 TX

Courtesy of NOUVADerm

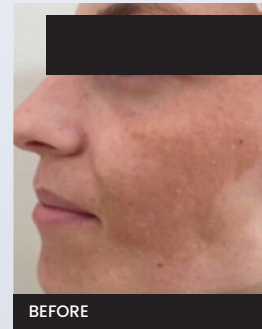


BEFORE

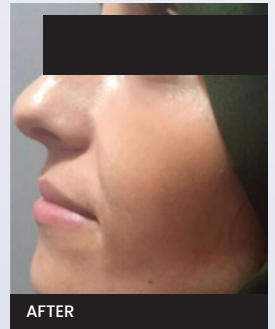

AFTER

Courtesy of Serkan Ozturk, MD

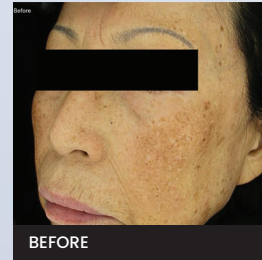

BEFORE



AFTER

Courtesy of Louis Dermatologic Clinic



BEFORE



AFTER

Courtesy of U. C. Y, MD,

NOUVA*Derm*

aestheticmanagementpartners.com/NOUVA_Derm

System Type	Thulium
Wavelength	1927 nm
Max Power	10 W
Scan Area	1 X 1 mm ~30c30 mm (1 x 1 mm step)
Dimensions	279(W) x 687(D) X 766(H) mm
Weight	30kg



www.aestheticmanagementpartners.com

9109 MACON ROAD
CORDOVA, TN 38016

(877) 267-2670
info@amplifimd.com